# ReSound Unite<sup>TM</sup> Mini Microphone

Using it with your child







### ReSound Up Smart™

### MAXIMIZE YOUR CHILD'S SPEECH AND LANGUAGE LEARNING

The ReSound Unite™ Mini Microphone can be used with ReSound Up Smart™ hearing aids. It is useful in many situations where understanding speech is particularly difficult. Some examples are in noisy environments or when a speaker isn't close to and directly in front of your child.

The ReSound Unite™ Mini Microphone is a small, portable streaming device that sends speech directly to children's hearing aids. It helps your child hear and understand speech better because it picks up sound where it's clearest – at its source – and transmits it via ReSound's proprietary 2.4 GHz digital wireless technology.



#### The ReSound Unite™ Mini Microphone:

- Provides up to 7 meters wireless connection
- · Clips discreetly onto clothing
- Sends sound from sources like smart phones, MP3 players and tablets.
- Is easy to use with an on/off button and volume control
- Has a multicolor LED light that shows battery, charging and pairing status



## MAXIMUM COMMUNICATION - ALSO IN THE CAR

With hearing aids alone, it's hard for your child to hear you in the car. There's lots of background noise, you can't sit face-to-face and you're relatively far away from each other. This can be frustrating for both of you. And when you can't sing songs together or chat about the passing landscape, your child misses out on an excellent opportunity for learning speechand-language.

The ReSound Unite™ Mini Microphone gives you and your child a strong communication connection that let's you talk, sing and learn together in the car. It also helps you drive safely – because you don't have to turn your head toward your child to make yourself understood.

### WHO IS THE RESOUND UNITE™ MINI MICROPHONE FOR AND WHEN IS IT HELPFUL?

The ReSound Unite™ Mini Microphone is for children of all ages with any degree of hearing loss. It helps them hear speech and language clearly in many challenging situations, including when they're:

- Riding in a car
- Being pushed in a buggy or pram
- · Walking beside you out of doors
- At preschool or daycare
- Visiting a museum
- At the grocery store
- In a classroom



